

Switch Stick Snow & Ice Grip

This snow & ice grip can be used with any walking stick.

To attach to a walking stick, undo the screws (you can use these smaller screws for your Switch Stick™). Other sticks may require the longer screws. For maximum protection of your stick against scratching you may wish to place material/felt/foam around the stick where the ice grip attaches.

Attach the ice grip so the 'spiked' base is flush with the bottom of the stick and, in effect, becomes the new base. Screw securely in place fastening the screws either side of the grip until completely tight, using a Phillips screwdriver. The stick is now ready for outdoor use.

When you return indoors, the two white levers can be squeezed together and the ice grip flipped up, positioning it along the stick, and thereby ensuring you don't damage any floors. You then simply use your stick as normal.


Please note we do NOT recommend users of walking sticks to go out in the ice as this is the easiest time to slip and fall. However, should you wish or need to go out in wintery conditions, the ice grip may offer some additional stability in the ice. Please take extra care. Switch Sticks cannot be held responsible for anyone who falls whilst using an ice grip.



Preparing to attach ice grip -Phillips screwdriver, ice grip & 2 shorter screws with Switch Stick™




Place body of ice grip under your Switch Stick™ making sure the base is flush with the ferrule

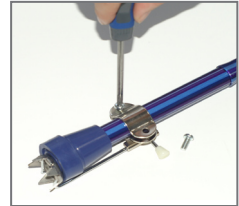

Screw the two parts of the ice grip together



Ice grip down for outdoor use

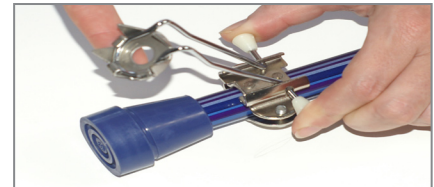

Squeezing white levers together to lift ice grip up



Ice grip up for indoor use